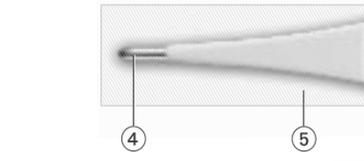
MT 550

IB MT 550 V-S10 V021, Revision Date: 2021-11-23

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- 1 ON/OFF button
- 2 Display
- 3 Battery compartment cover
- 4 Measuring sensor / measuring tip
- 5 Cleaning and disinfecting area (entire thermometer surface)

Important Safety Instructions

- Follow instructions for use. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.
- This device is only to be used for measuring human body temperature through oral, rectal or axillary. Do not attempt to take temperatures at other sites, such as in the ear, as it may result in false readings and may lead to injury.
- Do not use this device if you think it is damaged or notice anything unusual.
- We recommend cleaning this device according to the cleaning instructions before first use for personal hygiene.
- The minimum measurement time until the beep is heard must be maintained without exception! Consider that different measurement locations may require continued measuring even after the beep, see section «Measuring methods / Normal body temperature».
- Do not attempt rectal measurements on persons with rectal disorders. Doing so may aggravate or worsen the disorder.
- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.
- Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations. Keep a minimum distance of 3.3 m from such devices when using this device.
- Protect the device from impact and dropping!
- Avoid bending the thermometer probe more than 45°!
- Avoid ambient temperatures above 60 °C. NEVER boil this device!
- Use only the commercial disinfectants listed in the section «Cleaning and Disinfecting» to clean the device to avoid damage to the device.
- We recommend this device is tested for accuracy every two years or after mechanical impact (e.g. being dropped). Please contact your local MicroLife-Service to arrange the test.

WARNING: The measurement result given by this device is not a diagnosis! Do not rely on the measurement result only.

- Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

-  Read the instructions carefully before using this device.

-  Type BF applied part

Turning on the Thermometer

To turn on the thermometer, press the ON/OFF button 1; a short beep signals «thermometer ON». A display test is performed. All segments should be displayed.

The last measurement reading will be shown on the display 2 automatically for 2 seconds with the «M» icon.

Then at an ambient temperature of less than 32 °C, an «L» and a flashing «C» or «F» appear at the display field 2. The thermometer is ready for use now.

Function Test

Correct functioning of the thermometer is tested automatically each time it is turned on. If a malfunction is detected (measurement inaccuracy), this is indicated by «ERR» on the display, and a measurement becomes impossible. In this case, the thermometer must be replaced.

Using the Thermometer

Choose the preferred measuring method. When taking a measurement, the current temperature is continuously displayed and the «C» symbol flashes. If the beep is heard 5 times and the «C» is no longer flashing, the predictive end-temperature has been determined and the thermometer can be read now.

40 short beeps and a red LCD backlight alert the patient that he/she may have a temperature equal to or higher than 37.5 °C. Reference: Oral temperature. To achieve comparable results allow a 1 minute interval time between measurements.

To prolong the battery life, turn off the thermometer by briefly pressing the ON/OFF button 1. Otherwise the thermometer will automatically turn off after about 10 minutes.

Changing from °C to °F

With the thermometer turned off, press and hold the ON/OFF button 1 for 5 seconds until «M» disappears. Shortly press the ON/OFF button to switch between °C or °F. Wait 3 seconds till «C» or «F» starts blinking and press ON/OFF button again to confirm your selection. The thermometer is set to your selection for future use.

Measuring methods / Normal body temperature

► **In the armpit (axillary) / 34.7 - 37.3 °C**
Place the measuring sensor 4 under the arm into the center of the armpit so the tip is touching the skin and position the patient's arm next to the patient's body. This ensures that the room air does not affect the reading. Because the axillary takes more time to reach its stable temperature wait at **least 5 minutes**, regardless of the beep sound.

► **In the mouth (oral) / 35.5 - 37.5 °C**

Carefully insert the measuring sensor 4 of the thermometer 2 to 3 cm into the anal opening.

The use of a probe cover and the use of a lubricant is recommended. If you are unsure of this measurement method, you should consult a professional for guidance/training.

Approx. measuring time: 10 seconds!

► **In the anus (rectal) / 36.6 - 38.0 °C**

Carefully insert the measuring sensor 4 of the thermometer 2 to 3 cm into the anal opening.

The use of a probe cover and the use of a lubricant is recommended. If you are unsure of this measurement method, you should consult a professional for guidance/training.

Approx. measuring time: 10 seconds!

Cleaning and Disinfecting

For disinfection in home use environment, use a 70% Isopropyl alcohol swab or a cotton tissue moistened with 70% Isopropyl alcohol to wipe surface pollutants off the thermometer probe. Always start wiping from the end of the thermometer probe (approx. at the middle of the thermometer) towards the thermometer tip. Afterwards the entire thermometer (see number 5 in the drawing) should be immersed in 70% Isopropyl alcohol for at least 5 minutes (max. 24 hours). After immersion, let the disinfectant dry off for 1 minute before next use.

For professional use: Additionally, you may apply probe covers (available as accessory). Contact MicroLife for more details concerning further disinfectants for your thermometer model.

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